CURVE

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Safety First

The O₂ Curve is a high-performance particulate respirator. When used properly, it will prevent you from breathing in harmful airborne particles, allergens, viruses, and bacteria. Like any professional-grade equipment, you should treat the O₂ Curve with care to ensure your safety and long-lasting use of the mask.

Not for children

The O₂ Curve is only recommended for people over the age of 12.

Do not modify

The O₂ Curve is for use as-is. Modifying the mask will invalidate your warranty, potentially ruin the product, and may cause you bodily harm.

Proper cleaning

We recommend cleaning the mask with warm water and soap each time you change your filter. The O₂ Curve can be boiled and submersed.

Manufacturer

O2 Industries Inc.
5 Michael Street Unit 002
Kitchener ON N2G 1L7
Canada

Authorized European Representative

Simply Healthcare Solutions Limited
3 Rushton's Yard, Ashby de la Zouch,
Leicestershire, LE65 1AL,
United Kingdom

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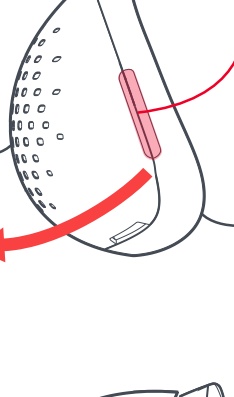
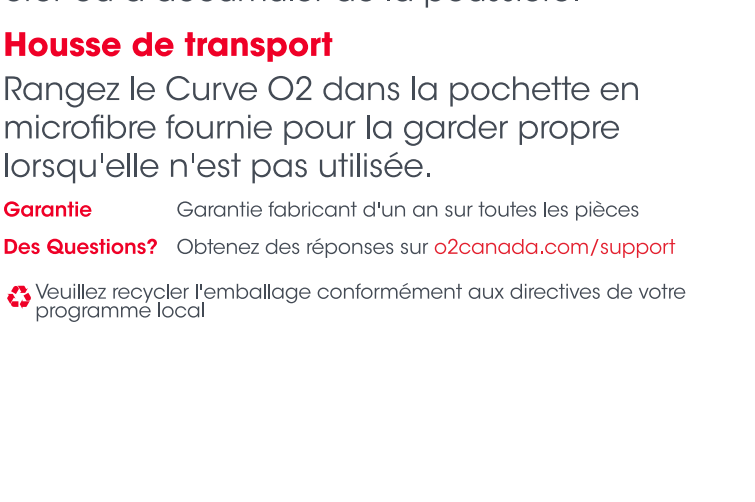
Filter Life

Original filters have an electrostatic charge and should be replaced at least every two weeks. Discoloration or dust build-up can be an indication that replacement is needed.

Carrying Pouch

Store your O₂ Curve in the provided microfiber pouch to keep it clean when not in use.

Warranty One year manufacturers warranty on all parts

Questions? Get answers at o2canada.com/support

Recycling Please recycle packaging according to your local program guidelines

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